

A photograph of an elderly couple walking hand-in-hand on a gravel path that stretches into the distance. The woman is on the left, wearing a dark denim jacket over a white shirt and light-colored pants. The man is on the right, wearing a grey cable-knit sweater and khaki pants. They are both smiling and looking at each other. The background consists of rolling green hills and a line of trees under a clear sky.

LATITUDE[®]
Patient Management System

**Boston
Scientific**
Delivering what's next.™

LATITUDE® Patient Management System

Staying more connected to your doctor

The LATITUDE® Patient Management system, which your doctor has prescribed, remotely connects patients to clinicians and helps to support more timely and more comprehensive cardiac care.



- Your implanted device needs to be checked regularly to review information that is stored in the device and to monitor settings to ensure that the device is programmed optimally to manage your heart conditions.
- These checks can happen in the clinic or from the comfort of your home using the LATITUDE Patient Management system.
- The system uses a Communicator that can sit at your bedside to monitor your cardiac device and transmit information through your regular landline phone to a secure website that only your healthcare support team can access.
- Your doctor can intervene sooner and make changes when they are needed. The LATITUDE Patient Management system gives you a higher standard of care.

LATITUDE[®] Patient Management System

- The LATITUDE Communicator collects information from your device in the convenience of your home.
- The Communicator can check your cardiac device in two ways:
 - Regularly scheduled follow-ups
 - Remote monitoring between follow-ups as determined by your physician
- Your information is sent to a secure website that only your healthcare support team can view.
- Your physician can access your information anytime, anywhere when needed.



LATITUDE[®] Communicator

Your Communicator will be shipped to your home in a brown Boston Scientific box.



Accept delivery, open the box and complete the set-up as soon as possible

What comes in the box?

- Your Communicator
- Phone Cord
- Power Supply
- Video Setup Guide
- Illustrated Communicator Setup Guide
- Patient Manual

Setup is simple.

- The LATITUDE Communicator plugs into your regular phone line and power outlet.
- After plugging it in, your Communicator will guide you through the setup process.


LATITUDE[®] Touch-Screen Wireless Communicator

For patients with a touch-screen, wireless Communicator

- The LATITUDE Patient Management System is easy to set up and use.
- A power cord and telephone landline is all you need to connect.

It is recommended that your LATITUDE Communicator stay near your bedside. The wireless Communicator automatically captures data from your device and sends the data through your regular landline phone to a secure website where only your healthcare support team can view your information.

The monitoring of your heart health and specific device information will be done on a scheduled basis as determined by your physician.

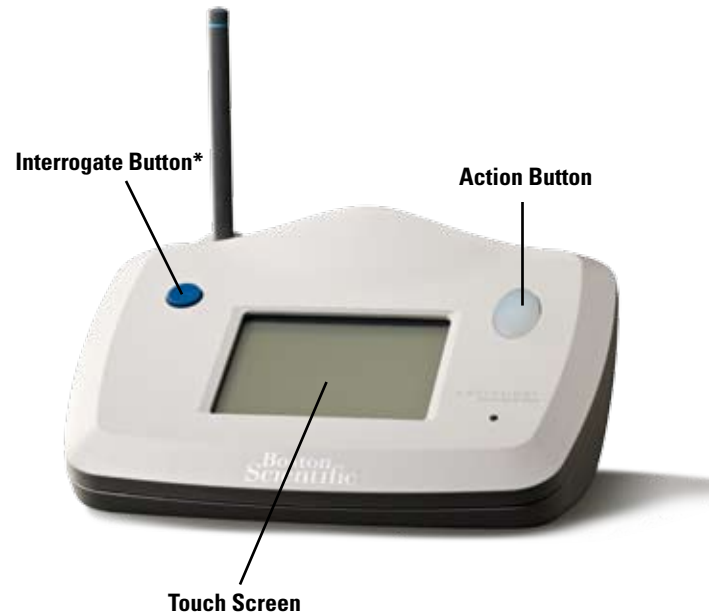

LATITUDE® Push Button Wireless Communicator

For patients with a push button wireless Communicator

- The LATITUDE Patient Management System is easy to set up and use.
- A power cord and telephone landline is all you need to connect.

It is recommended that your LATITUDE Communicator stay near your bedside. The wireless Communicator automatically captures data from your device and sends the data through your regular landline phone to a secure website where only your health care support team can view your information.

The monitoring of your heart health and specific device information will be done on a scheduled basis as determined by your physician.



LATITUDE® Wanded Communicator

For patients with a wanded Communicator

It is recommended that your LATITUDE Communicator stay near your bedside. The Communicator captures data from your device and sends the data through your regular landline phone to a secure website where only your healthcare support team can view your information.

The wanded Communicator is easy to use. The blinking white action button will remind you when it's time to transmit your information.

Simply hold the wand over your device, then push the blue interrogate button to send your information. On average, it takes 10-15 seconds to complete but may take more time if additional information is needed.

The Communicator will provide an on-screen confirmation that your information was successfully sent.



LATITUDE® Patient Support



Contact LATITUDE Patient Support if you or your family has questions regarding:

- Setup/Operation
- Other LATITUDE related topics

The LATITUDE Patient Support hotline is available to you by dialing 1.866.484.3268. This live, dedicated, multi-lingual service is available to you at no additional charge whenever you may need it.

LATITUDE® Patient Management System from Boston Scientific CRM

The LATITUDE Patient Management system is used to remotely communicate with a compatible pulse generator device from BSC CRM and send data to a central database. The LATITUDE system is contraindicated for use with any pulse generator other than a device from BSC CRM.

The LATITUDE system is designed to tell your doctor within 24 hours if alert conditions are detected by the Communicator. However, alert notification cannot occur if:

- The Communicator is unplugged or is not able to connect to the LATITUDE system through an active phone line.
- Your device and the Communicator cannot establish and complete a communication session. This session must be initiated by you if you have a device that uses inductive telemetry (Communicator that has a wand).
- The Communicator becomes damaged or it malfunctions.

Up to two weeks may go by before the LATITUDE system detects the events mentioned above, and additional time may be required for notification and resolution of the condition.

The wand and wireless Communicator uses a radio frequency (RF) communication system to communicate with an optional weight scale and blood pressure monitor. This communication can be disrupted by electromagnetic interference. Avoid placing your Communicator next to or in the immediate vicinity of other wireless products and sources of electromagnetic energy. The wireless Communicator uses RF to also send and receive signals from the implanted device (RF enabled devices only). Using the blue Interrogate button more than as prompted by your Communicator or as instructed by your physician may lead to a decrease in the battery life of your implanted device. Your communicator is designed to be used in the continental US, Alaska, Hawaii, and Puerto Rico. These devices are available by prescription only.

(Rev. I)

LATITUDE® Compatible Devices

Wanded Communicator (Model 6481)



Push Button Wireless Communicator (Model 6482)



Touch Screen Wireless Communicator (Model 6476)



H115 CONTAK® CD 2
 H119 CONTAK CD 2 HE
 H135 CONTAK RENEWAL®
 H170 CONTAK RENEWAL 3
 H175 CONTAK RENEWAL 3
 H177 CONTAK RENEWAL 3 HE
 H179 CONTAK RENEWAL 3 HE
 1850 VENTAK PRIZM® VR
 1852 VENTAK PRIZM VR HE
 1857 VENTAK PRIZM VR HE
 1860 VENTAK PRIZM 2 VR
 1851 VENTAK PRIZM DR
 1853 VENTAK PRIZM DR HE
 1858 VENTAK PRIZM DR HE
 1861 VENTAK PRIZM 2 DR
 T135 VITALITY® DS VR
 T125 VITALITY DS DR
 T127 VITALITY EL DR
 T180 VITALITY HE
 T165 VITALITY 2 DR
 T167 VITALITY 2 EL DR
 T175 VITALITY 2 VR
 T177 VITALITY 2 EL VR

H210 CONTAK RENEWAL 3 RF
 H215 CONTAK RENEWAL 3 RF
 H217 CONTAK RENEWAL 3 RF HE
 H219 CONTAK RENEWAL 3 RF HE
 E030 CONFIENT™ RF HE
 H220 LIVIAN™ RF
 H225 LIVIAN RF
 H227 LIVIAN RF HE
 H229 LIVIAN RF HE

N118 COGNIS™ RF HE
 N119 COGNIS RF HE
 E102 TELIGEN™ VR RF HE
 E110 TELIGEN DR RF HE

**Boston
Scientific**

Delivering what's next.™

Cardiac Rhythm Management

Boston Scientific Corporation
 4100 Hamline Avenue North
 St. Paul, MN 55112-5798 USA
 Tel: 651.582.4000 Fax: 651.582.4166
 Medical Professionals:
 1.800.CARDIAC (227.3422)
 Patients and Families:
 1.866.484.3268

www.bostonscientific.com

Copyright © 2008 by
 Boston Scientific Corporation
 or its affiliates. All rights reserved.

C7-406-1008